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WORK DETAILS

Engineer Name: SHUYA MIYAZAKI

Event Type: Predictive / Preventative Maintenance

Device Status: Pass

Service Activity: Preventative Maintenance

EQUIPMENT DETAILS

Product Name: EPIQ 7 Ultrasound System

Serial Number: USO1580925

Product Number: 795200

Installed Product: 67704928

UDI:

	Min	Max	UoM	Measurement	Result	Comments
Planned Maintenance						
System Information						
Software Version					7.0.5	
Hardware Version					B.1	
Inspection & Cleaning						
Query Customer Performance Issues					Pass	
Inspect Exterior Periph. & Power Cords.					Pass	
Inspect & Clean Trackball					Pass	
Inspect & Clean Monitor					Pass	
Inspect/Clean Control Panels & Keyboard					Pass	
Inspected and cleaned the transducer connector assembly (transducer and system side)?					Yes	
Footswitch, Key- & Remote Control Pad					Not Installed	
Inspect Casters & Brakes Operation					Pass	
Clean/Check System Filters & Fans					Pass	
Check Articulation motion.(monitor, etc)					Pass	
Check System Battery					Pass	
Performance & Peripherals						
Execute Full Set of System Diagnostics					Pass	
Check, Download & Clear Logs					Pass	
Perform Hard-disk Maintenance					Pass	
Backup of System Settings and Configs.					Pass	

	Min	Max	UoM	Measurement	Result	Comments
Planned Maintenance						
Performance & Peripherals						
Check Operation All User Ctrls. & Lights					Pass	
Check ECG Cable and Trace on Screen					Pass	
Check Operation of Major Applications					Pass	
Test Printers					Pass	
Test VCR Operation & Clean					Not Installed	
Check Operation Dig. Recording Devices					Pass	
Confirm remote connectivity					Not Installed	
System Safety General						
Protective Earth resistance (<= 200 mOhm)		200	mOhm	53	Pass	
Earth Leakage Current. NC, Highest val. (<= 500 µA)		500	µA	175.9	Pass	
Earth Leakage Current. SF, Highest val. (<= 1000 µA)		1000	µA	339	Pass	
ECG Body - BF						
ECG Lead Leakage Current "Norm Cond"		100	µA	0.1	Pass	
ECG LL Curr. Highest SF or 2SF reading		500	µA	1.6	Pass	
ECG Lead Isolation, ISO/MAP "Sink"		500	µA	2	Pass	
Transducer Body - BF						
Transducer model					L12-3	
Transducer serial number					B1LVYF	
Transducer Cables, Connectors & Lens					PASS	
XDR Leakage Current "Normal Condition" (<= 100 µA)		100		0.1	Pass	
XDR Leak Curr. Highest SF or 2SF reading" (<= 500 µA)		500		1.1	Pass	
XDR Isolation, ISO/MAP "Sink" (<= 500 µA)		500		2.4	Pass	
Run Transducer Test					PASS	
Transducer Body - BF						
Transducer model					C9-2	
Transducer serial number					B1KXPJ	
Transducer Cables, Connectors & Lens					PASS	
XDR Leakage Current "Normal Condition" (<= 100 µA)		100		0.1	Pass	
XDR Leak Curr. Highest SF or 2SF reading" (<= 500 µA)		500		2.2	Pass	
XDR Isolation, ISO/MAP "Sink" (<= 500 µA)		500		4.9	Pass	
Run Transducer Test					PASS	
Transducer Heart - CF						
Transducer model					S9-2	
Transducer serial number					B3G9RZ	
Transducer Cables, Connectors & Lens					PASS	

	Min	Max	UoM	Measurement	Result	Comments
Planned Maintenance						
Transducer Heart - CF						
XDR Leakage Current "Normal Condition" (<= 10 µA)		10		0.1	Pass	
XDR Leak Curr. Highest SF or 2SF reading (<= 50 µA)		50		1.5	Pass	
XDR Isolation, ISO/MAP "Sink" (<= 50 µA)		50		3.4	Pass	
Run Transducer Test					PASS	
Transducer Heart - CF						
Transducer model					S8-3	
Transducer serial number					B1IVCG	
Transducer Cables, Connectors & Lens					PASS	
XDR Leakage Current "Normal Condition" (<= 10 µA)		10		0.1	Pass	
XDR Leak Curr. Highest SF or 2SF reading (<= 50 µA)		50		0.7	Pass	
XDR Isolation, ISO/MAP "Sink" (<= 50 µA)		50		1.7	Pass	
Run Transducer Test					PASS	
Transducer Heart - CF						
Transducer model					X5-1	
Transducer serial number					B37PJJ	
Transducer Cables, Connectors & Lens					PASS	
XDR Leakage Current "Normal Condition" (<= 10 µA)		10		0.1	Pass	
XDR Leak Curr. Highest SF or 2SF reading (<= 50 µA)		50		1.5	Pass	
XDR Isolation, ISO/MAP "Sink" (<= 50 µA)		50		3.5	Pass	
Run Transducer Test					PASS	
Transducer TEE - BF						
Transducer model					X8-2t	
Transducer serial number					F03DY8	
Inspect Transducer Cable					PASS	
XDR Leakage Current "Normal Condition" (<= 300 µA)		300	µA	0.1	Pass	
XDR Leakage Current "Single Condition" (<= 500 µA)		500	µA	30.3	Pass	
XDR Isolation, ISO/MAP "Sink" (<= 500 µA)		500	µA	81.2	Pass	
Inspect TEE Xdcrs & Controls					PASS	
Check Sheath condition					PASS	
Check Nose Piece for Damage					PASS	
Check Articulation Deflection Cover					PASS	
Check Handle / Connector for Damage					PASS	
Check TEE control, Tip Deflection					PASS	
Verify TEE Handle & Connector for Fluid					PASS	
Verify Control Knob Locks					PASS	

Min	Max	UoM	Measurement	Result	Comments
-----	-----	-----	-------------	--------	----------

Planned Maintenance

Transducer TEE - BF

Review TEE Care & Handling with Customer

PASS

Philips Representative Signature:

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